

Optimized Device Performance Guide

CAAB + CAZB Linear Stapler with knife

1. Preparation



Open the packaging from the marked side and take out the stapler.



Remove the red protector by pressing the thumb "PUSH".

2. Place tissue in and Reposition



Place tissue between the jaws (The retaining pin can be pushed manually or automatically)



Squeeze the firing handle half way, firing handle will get locked with the jaws half closed. In this position you can do the final adjustment of the tissue without unnecessary trauma.

3. Close the jaw



Squeeze the firing handle fully and release it to make sure it moves back to the original position. In this position the jaws are fully closed and the tissue is compressed.

4. Firing




Continuously squeeze the firing handle until it stops so as to complete stapling and cutting. To make sure you have fully fired the instrument you need to feel a stop at the firing trigger and not be able to keep firing.

5. Open the jaw and Reset



Press release button and the jaws will open.

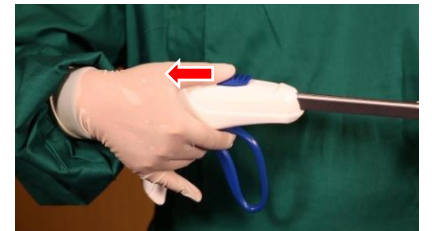


Pull back the retaining pin knob to the original position .

6. Remove used RU



To remove the reloading unit, grasp the finger pads at the top and pull it straight upward from the jaws.

7. Load a new RU

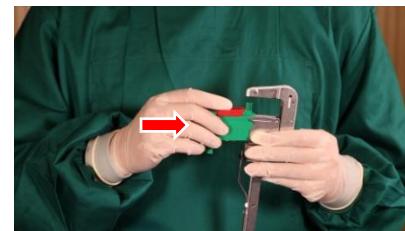

To reload the instrument, make sure RU groove is corresponding with cartridge sliding splint. Insert the RU into the metal cartridge housing and push firmly downward .



Then remove the red protector.

